- STEP 1:** Select site.
STEP 2: Wash site using a plain soap, dry, and then clean with alcohol wipe. Allow site to air dry.
STEP 3: Peel lid completely off Sensor Pack. Unscrew cap from Sensor Applicator.

CAUTION: Sensor codes must match on Sensor Pack and Sensor Applicator. Do NOT use if damaged or already opened.

- STEP 4:** Line up dark marks on Sensor Applicator and Sensor Pack. On a hard surface, press down firmly on Sensor Applicator until it comes to a stop.

- STEP 5:** Lift Sensor Applicator out of Sensor Pack.
STEP 6: Sensor Applicator is ready to apply Sensor.

CAUTION: Sensor Applicator now contains a needle. Do not touch inside Sensor Applicator or put it back into Sensor Pack.

- STEP 7:** Place Sensor Applicator over site. Push down firmly to apply Sensor.

CAUTION: Do NOT push down on Sensor Applicator until placed over prepared site to prevent unintended results or injury.

- STEP 8:** Gently pull Sensor Applicator away.
STEP 9: Make sure Sensor is secure. Discard the used Sensor Pack and Sensor Applicator. See *Disposal* section in the User's Manual.

To use your Sensor, refer to all the labelling provided with the Reader Kit. If using FreeStyle LibreLink app or FreeStyle LibreLink special edition app, refer to the User's Manual in the App.

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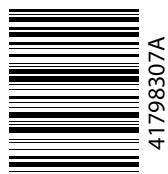
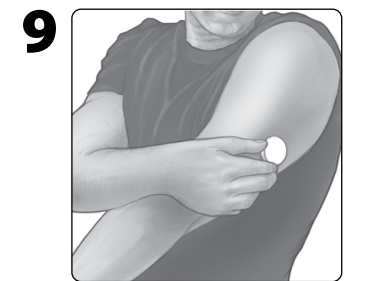
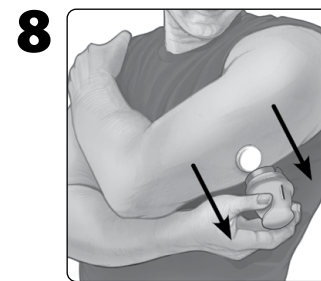
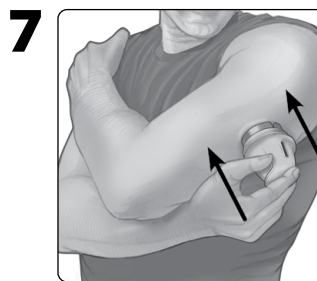
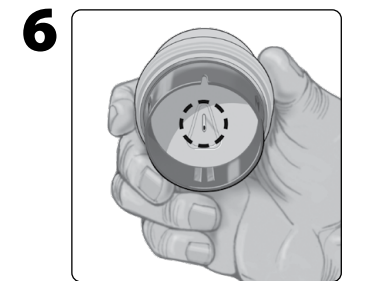
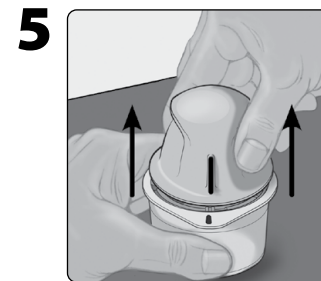
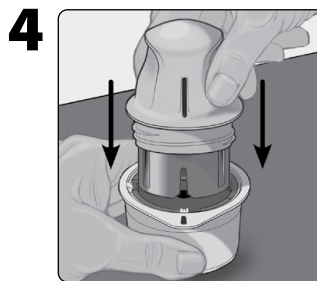
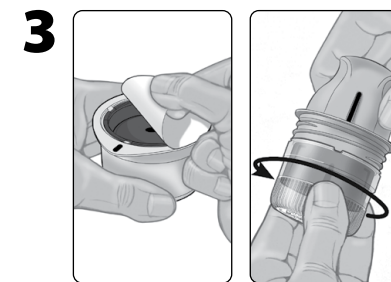
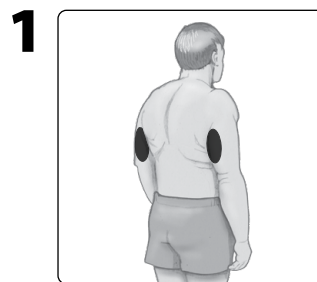
41798307A

Hereby, Abbott Diabetes Care Ltd. declares that the radio equipment type FreeStyle Libre 2 Sensor is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: www.diabetescare.abbott/doc

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FreeStyle Libre 2 Sensor

READ ALL THE LABELLING PROVIDED WITH THE READER KIT BEFORE HANDLING THE SENSOR KIT PARTS. IF USING FREESTYLE LIBRELINK APP OR FREESTYLE LIBRELINK SPECIAL EDITION APP, REFER TO THE USER'S MANUAL IN THE APP.

For use with: FreeStyle Libre Reader, FreeStyle Libre 2 Reader, FreeStyle LibreLink app and FreeStyle LibreLink special edition app. (Not all products are available in all countries.)

Indications for Use

The FreeStyle Libre 2 Flash Glucose Monitoring System Sensor ("Sensor"), when used with a compatible device, is indicated for measuring interstitial fluid glucose levels in people (age 4 and older) with diabetes mellitus, including pregnant women. The Sensor and compatible device are designed to replace blood glucose testing in the self-management of diabetes, including dosing of insulin.

The indication for children (age 4 - 12) is limited to those who are supervised by a caregiver who is at least 18 years of age. The caregiver is responsible for managing or assisting the child to manage the Sensor and compatible device and also for interpreting or assisting the child to interpret Sensor glucose readings.

CONTRAINDICATION:

The Sensor must be removed prior to Magnetic Resonance Imaging (MRI).

WARNING:

- The Sensor contains small parts that may be dangerous if swallowed.
- The FreeStyle Libre 2 Sensor can be used with the FreeStyle Libre Reader but the FreeStyle Libre Reader will NOT issue alarms.

Note: For information on what your Sensor glucose reading means, refer to the User's Manual of the device that you are using with your Sensor.

CAUTION:

- On rare occasions, you may get inaccurate Sensor glucose readings. If you notice your readings are not correct or are inconsistent with how you feel, perform a blood glucose test on your finger to confirm your glucose and check to make sure your Sensor has not come loose. If the problem continues or if your Sensor is coming loose, remove the current Sensor and apply a new one.
- Intense exercise may cause your Sensor to loosen due to sweat or movement of the Sensor. If your Sensor comes loose, you may get no readings or unreliable readings, which may not match how you feel. Follow the instructions to select an appropriate application site.
- Do not reuse Sensors. The Sensor and Sensor Applicator are designed for single use. Reuse may result in no glucose readings and infection. Not suitable for re-sterilization. Further exposure to irradiation may cause inaccurate results.
- The Sensor Pack and Sensor Applicator are packaged as a set and have the same Sensor code. Check that the Sensor codes match before using your Sensor Pack and Sensor Applicator. Sensor Packs and Sensor Applicators with the same Sensor code should be used together or your Sensor glucose readings may be incorrect.
- Some individuals may be sensitive to the adhesive that keeps the Sensor attached to the skin. If you notice significant skin irritation around or under your Sensor, remove the Sensor and stop using the Sensor. Contact your health care professional before continuing to use the Sensor.

Description

The Sensor initially comes in two parts: one part is in the Sensor Pack and the other part is in the Sensor Applicator. Follow the instructions to prepare and apply the Sensor on the back of your upper arm. The Sensor has a small, flexible tip that is inserted just under the skin. The Sensor can be worn for up to 14 days.

Sensor Kit Parts

Sensor Pack

Used with Sensor Applicator to prepare Sensor for use.

Sensor Applicator

Applies Sensor to body.

User Safety, Storage and Handling

- The Sensor Pack is sterile unless opened or damaged.
- Store the Sensor Kit between 4°C and 25°C. While you don't need to keep your Sensor Kit in a refrigerator, you can as long as the refrigerator is between 4°C and 25°C.
- Your Sensor has been tested to withstand immersion into one metre (3 ft) of water for up to 30 minutes. It is also protected against insertion of objects > 12 mm diameter. (IP27)
- Do not freeze. Do not use if expiry date has passed.

Disposal

Refer to User's Manual for disposal instructions.

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FreeStyle, Libre, and related brand marks are marks of Abbott. Other trademarks are the property of their respective owners.
Abbott B.V.